

Wearing Power Paws

Did I order the right size?

Looks can be deceiving! The product you order may appear to be too small for your dog. Despite the appearance, please try one on. You may be surprised at the amount they stretch.

If the size is right, Power Paws will stay on when playing, climbing stairs, walking etc. A proper fit is a snug, stretched sock allowing the elastic to hold the sock in place. To be sure, even if it looks tight, if you can put a finger in the sock, they aren't too tight and the fit is good. If the sock is too small, you will not be able to get it on. If the sock is too large, it will rotate (so the grip rolls to the side or top of the sock), or it may fall off completely.

Will the Power Paws stay on?

Absolutely, if they are the right size! The cotton material was chosen for comfort. The elastic ensures a snug, custom fit that stays on. Because Power Paws are so comfortable, dogs generally do not try to take Power Paws off.

Is it normal for a dog to shake his paws or stand motionless when the Power Paws are first put on?

For many dogs, yes. If your dog is not used to having socks or shoes on his feet, he may react this way. It is temporary! Most dogs adapt to Power Paws within minutes. It is suggested that you call your dog to come to you or throw them their favourite toy to encourage movement. Dogs learn quickly, especially if Power Paws are needed for traction or mobility, that Power Paws = mobility! Do not be surprised if your dog WANTS them on.

Washing and Care Instructions

You may find that with the new mobility your pet has while wearing Power Paws, he or she won't stop at the door long enough for you to remove them before venturing outside! When your Power Paws get dirty, just throw them in the washing machine! It is recommended that a gentle wash cycle with cold water be used in order to avoid shrinkage. Dry on low or medium heat and your Power Paws will come out clean and as good as new.

(If you are trying to shrink your Power Paws just a bit for a better or custom fit, wash them in warm water, then dry them on a low heat).

How long do Power Paws last?

A set of Power Paws can be worn daily for several months without any problem. They can be washed and dried many times (a delicate cycle with cold water and drying on low heat). Power Paws should ideally be worn for indoor use only.